SR2S, SR4S

en MEDICAL ADHESIVE REMOVER INSTRUCTIONS FOR USE

INTENDED USE: Formulated to remove medical adhesive from skin and appliances.
INSTRUCTIONS: Spray remover onto adhesive residue or gauze pad and wipe adhesive away. You may also spray remover around the edges of an appliance to aid in its removal from skin. Test on an inconspicuous area before widespread use.
WARNING: Flammable. Do not spray on a flame or any incandescent material, electrical appliance or source of heat. Keep away from sources of ignition – No Smoking. Keep out of the reach of children. Avoid spraying into eyes. Do not apply to open wounds or mucous membranes. Discontinue use if irritation occurs.
NOTES: Contact a healthcare professional for help with skin irritation. In case of serious incident, please contact the manufacturer or local representative, and your local competent authority.
DISPOSAL: Dispose of container/contents in accordance with local/national regulations.
INGREDIENTS: Octamethyltrisiloxane solvent.

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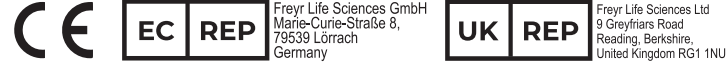
Real results. Real people. Real confidence. Tag us! ***#SkinisterMedical***

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